

DRIVE DEVILBISS® IGO AMBULATORY

PORTABLE OXYGEN CONCENTRATOR

Preventive Maintenance

Weekly:



Clean Air Filter. Replace as needed and between patients.

Periodically:



Clean and replace the cannula and tubing as instructed by the manufacturer and your equipment provider

Every 3 Years:



Replace internal intake filter. We also recommend changing between residents.

When changing residents:



Clean and disinfect the exterior covers of the device and rechargeable battery between patient use and as needed.



#7DJ14

drive | **DeVilbiss**
HEALTHCARE

- Integrated PulseDose® oxygen conserving technology recognized for clinically effective pulse-dose oxygen delivery
- Built-in OSD® (oxygen sensing device) ensures accurate oxygen delivery and reduced periodic maintenance schedule
- Easy-to-read battery life gauge with fast battery charge time – 2 to 4.5 hours dependent upon the flow setting

Specifications:	
Flow Rate:	1.0 - 3.0 lpm continuous flow, 1-6 pulse settings
O ₂ Purity %:	91% ±3
Sound Level:	40 dBA
Battery Life:	1.6 to 4.0 hrs on continuous flow, 3.0 to 5.4 hrs on pulse settings 1-6
Dimensions:	15" W x 11" D x 8" H
Weight:	19 lbs with battery
Warranty:	3 years on concentrator, 1 year on battery