

RESPIRONICS MILLENNIUM M10

10-LITER OXYGEN CONCENTRATOR



#75876

PHILIPS
RESPIRONICS

- Optional alarms for purity and flow rate
- Large humidifier bottle area
- Filters replaced every 2 years

Specifications:

Flow Rate:	10 lpm (1-liter increments)
O ₂ Purity %:	92% \pm 4% at 8 - 10 lpm
	94% \pm 2% at 3 - 7 lpm
	92% \pm 4% at 1 - 2 lpm
O ₂ Pressure:	10 - 30 psi
Power Consumption:	600W
Sound Level:	less than 50 dB
Dimensions:	19"W x 13"D x 27"H
Weight:	53 lbs.
Warranty:	1 year

Preventive Maintenance

Once a Week:



Clean the cabinet filter.

Once Every 2 Years:



Check and, if necessary, change the inlet HEPA filter and pre-inlet filter. We also recommend changing between residents.



Check and, if necessary, change the bacteria filter. We also recommend changing between residents.



Check and, if necessary, change the long-life filter. We also recommend changing between residents.



Verify the oxygen purity, flow rate and pressure of your concentrator by using an oxygen analyzer.

When changing residents:



Discard oxygen tubing, cannula/mask, oxygen outlet connector, and humidifier bottle. Replace cabin filter. Clean concentrator cabinet. Inspect plugs, cords and components. Check oxygen concentration.