

# INVACARE® PLATINUM™ 10

## 10-LITER OXYGEN CONCENTRATOR



#80911



Yes, you can:

## Preventive Maintenance

### Once a Week:



Clean the cabinet filter.

### Once a Year:



Check and, if necessary, change the inlet compressor filter. We also recommend changing between residents.



Check and, if necessary, change the bacteria filter. We also recommend changing between residents.



Verify the power-loss alarm is working. Let the concentrator run for 1 minute then unplug the power cord. An alarm should sound, indicating it is working properly.

### Once Every 3 Years:



Verify the oxygen purity, flow rate and pressure of your concentrator by using an oxygen analyzer.

### When changing patients:



Discard oxygen tubing, cannula/mask, oxygen outlet connector, and humidifier bottle. Replace cabin filter. Clean concentrator cabinet. Inspect plugs, cords and components. Check oxygen concentration.

- Trusted and reliable brand
- Optional alarms for flow rate and purity
- 3-year warranty

Specifications:	
Flow Rate:	2 - 10 lpm
O <sub>2</sub> Purity %:	87% - 94%
O <sub>2</sub> Pressure:	9 ±0.5 psi
Power Consumption:	585W
Sound Level:	58 dB
Dimensions:	18 <sup>3</sup> / <sub>8</sub> "W x 14 <sup>3</sup> / <sub>8</sub> "D x 26 <sup>3</sup> / <sub>8</sub> "H
Weight:	54 lbs.
Warranty:	3-year limited