

DRIVE DEVILVISS® COMPACT 10L

10-LITER OXYGEN CONCENTRATOR

Preventive Maintenance

Once a Week:



Clean the cabinet filter.

Once Every 2 Years:



Check and, if necessary, change the inlet compressor filter. We also recommend changing between residents.



Check and, if necessary, change the bacteria filter. We also recommend changing between residents or every 17,250 hours.



Verify the power-loss alarm is working. Let the concentrator run for 1 minute then unplug the power cord. An alarm should sound, indicating it is working properly.

Once Every 3 Years:



Verify the oxygen purity, flow rate and pressure of your concentrator by using an oxygen analyzer.

When changing residents:



Discard oxygen tubing, cannula/mask, oxygen outlet connector, and humidifier bottle. Replace cabin filter. Clean concentrator cabinet. Inspect plugs, cords and components. Check oxygen concentration.



#4HR15

drive | **DeVilbiss**
HEALTHCARE

- Several audible and visual alarms: high pressure, high/low flow, low oxygen, power failure
- Oxygen Sensing Device (OSD) standard
- 3-year limited warranty

Specifications:

Flow Rate:	2 - 10 lpm
O ₂ Purity %:	87% - 96%
O ₂ Pressure:	20 +/- 1.0 psi
Power Consumption:	639W
Sound Level:	55 dB
Dimensions:	13½"W x 12"D x 24½"H
Weight:	42 lbs.
Warranty:	3-year limited