

# RESPIRONICS SIMPLYGO

## PORTABLE OXYGEN CONCENTRATOR



#A0065

**PHILIPS**  
**RESPIRONICS**

- Nonclinical style and ease of use make it appealing to residents
- Easy-to-remove battery for quick maintenance
- Multiple alarm types make troubleshooting easy

## Preventive Maintenance

### Once Every 1 - 2 Years:



Check and, if necessary, change the inlet filter. We also recommend changing between residents.

### When changing residents:



Discard oxygen tubing, cannula/mask, oxygen outlet connector, and humidifier bottle. Replace cabin filter. Clean concentrator cabinet. Inspect plugs, cords and components. Check oxygen concentration.

Specifications:	
Flow Rate:	0.5 - 2.0 continuous flow 1 - 6 lpm pulse in 0.5 lpm increments
O <sub>2</sub> Purity %:	87% - 96%
O <sub>2</sub> Pressure:	3 psi
Power Consumption:	150W while charging 120W while not charging
Sound Level:	≤ 43 dB at pulse setting 2
Battery Life:	3½ hours / 7 hours with extra battery
Dimensions:	10"W x 6"D x 11½"H
Weight:	10 lbs.; 11.5 lbs. with extra battery
Warranty:	2 years