

# CAIRE® COMPANION 5™

## 5-LITER OXYGEN CONCENTRATOR

### Preventive Maintenance

#### Once Every 2 Years:



Check and, if necessary, change the air intake filter. We also recommend changing between residents.



Check and, if necessary, change the HEPA filter. Direct Supply suggests changing filter between residents.



Verify the oxygen purity, flow rate and pressure of your concentrator by using an oxygen analyzer.

#### When changing residents:



Discard oxygen tubing, cannula/mask, oxygen outlet connector, and humidifier bottle. Replace cabin filter. Clean concentrator cabinet. Inspect plugs, cords and components. Check oxygen concentration.



#C5319



- Maintains higher O<sub>2</sub> purity as flow rate increases
- Minimizes “exhale” noise for improved resident experience
- Reduces power consumption by 15% when running at 2 lpm or below
- Optional alarm screen displays error codes to specify technical issues
- Only two filters to change every 2 years – no weekly preventive maintenance

Specifications:	
Flow Rate:	0.5 - 5.0 lpm
O <sub>2</sub> Purity %:	90% +5.5 / -3%
Outlet Pressure:	4.6 psi
Power Consumption:	250W at 2 lpm, 350W at max.
Sound Level:	40 dB
Dimensions:	12½"W x 13½"D x 21½"H
Weight:	36 lbs.
Warranty:	3 years*