

# CAIRE® FREESTYLE® COMFORT®

PORTABLE OXYGEN CONCENTRATOR

## Preventive Maintenance

### Once a Week:



Clean 2 air intake filters. Your oxygen provider may advise cleaning more frequently depending on operating conditions.

### Every 3-6 months:



Replace the nasal cannula and tubing (every 3 months if daily use), or as recommended by the cannula manufacturer or your oxygen provider.

### When changing residents:



Discard oxygen tubing, cannula/mask, oxygen outlet connector, and humidifier bottle. Replace cabin filter. Clean concentrator cabinet. Inspect plugs, cords and components. Check oxygen concentration.



#6YR11

CAIRE®

- UltraSense sensitive breath technology
- Ergonomic shape
- Color display and glow in the dark keypad
- Long battery life up to 8 hours
- Internal filters designed to last the life of the device—no replacement needed

Specifications:	
Flow Rate:	1 - 5 pulse settings
O <sub>2</sub> Purity %:	90% +5.5/-3%
Sound Level:	40 dBA at pulse setting 2
Battery Life:	Up to 4 hours with 8-cell battery pack. Up to 8 hours with 16-cell battery pack
Dimensions:	7 <sup>3</sup> / <sub>10</sub> "W x 3 <sup>1</sup> / <sub>10</sub> "D x 10"H
Weight:	5 lbs.
Warranty:	3 years on concentrator, 1 year on battery and accessories