

# INVACARE® PERFECTO2™ V

5-LITER OXYGEN CONCENTRATOR



#37239



Yes, you can.®

- Industry-leading brand
- Built-in alarms for O<sub>2</sub> purity and flow rate come standard
- Economical price point

#### Specifications:

Flow Rate:	0.5 - 5.0 lpm
O <sub>2</sub> Purity %:	87% - 95.6%
Outlet Pressure:	5 psi
Power Consumption:	325W
Sound Level:	43 dB
Dimensions:	13"W x 11½"D x 23"H
Weight:	39 lbs.
Warranty:	3-year limited



Watch a short video and see how truly fast and easy these maintenance steps are!

Visit [DirectSupply.net/oxygen](http://DirectSupply.net/oxygen)

## Preventive Maintenance

### Once a Week:



Clean the cabinet filter.

### Once a Year:



Check and, if necessary, change the inlet compressor filter. We also recommend changing between residents.



Check and, if necessary, change the bacteria filter. We also recommend changing between residents.



Verify the power-loss alarm is working. Let the concentrator run for 1 minute then unplug the power cord. An alarm should sound, indicating it is working properly.

### Once Every 3 Years:



Verify the oxygen purity, flow rate and pressure of your concentrator by using an oxygen analyzer.

### When changing residents:



Discard oxygen tubing, cannula/mask, oxygen outlet connector, and humidifier bottle. Replace cabin filter. Clean concentrator cabinet. Inspect plugs, cords and components. Check oxygen concentration.