

# INVACARE® PLATINUM

## PORTABLE OXYGEN CONCENTRATOR



#29V89,  
#29V90



Yes, you can.

- Semi-pulse technology adjusts unit automatically to deliver the right amount of oxygen
- Easy-to-use four-button interface
- Dual intakes maximize heat dissipation
- Reinforced seals on screen and battery door, integrated drainage channels, and downward-facing ports and vents that meet IP22 water resistance standards for exposure up to 10 minutes

#### Specifications:

Flow Rate:	1 - 4 pulse settings
O <sub>2</sub> Purity %:	87.0% - 95.6%
Sound Level:	≤ 40 dBA on setting 2
Battery Life:	5 hours/10 hours with extra battery (on setting 1)
Dimensions:	7.5"W x 3.88"D x 9.45"H increments
Weight:	4.98 lbs., including unit and battery (not including carrying case)
Warranty:	5 years limited on concentrator, 2 years limited on compressor, 1 year limited on sieve, 1 year limited on batteries

## Preventive Maintenance

### Weekly:



Record date of service and elapsed hours. Clean the air intake filter located on the carry bag at least once each week. Use a vacuum cleaner or nylon brush to clean dust/debris from the air intake filter screen. Inspect cabinet, electrical cords and AC adapter for damage.

### Periodically:

Clean the concentrator cabinet. Turn off the concentrator and use a damp cloth or sponge with a mild detergent to gently clean the exterior case. Towel dry or allow to completely air-dry before returning to carry bag and prior to operating.

### Every 3 Years:

Check oxygen purity. Perform functional test.

### When changing residents:

Dispose and replace cannula and tubing. Clean filters and cabinet. Check inlet filter screen for damage. Check oxygen purity. Check/replace outlet filter. Perform functional test.