

AIRSEP FREESTYLE PORTABLE

PORTABLE OXYGEN CONCENTRATOR

Preventive Maintenance

Once a Week:



Clean and, if necessary, change the intake (gross particle) filter. We also recommend changing between residents. Your equipment provider may advise you to clean it more often, depending on your operating conditions.

Periodic Intervals:



Test the oxygen purity with each new resident and at periodic intervals determined by the equipment provider.

When changing patients:



Discard oxygen tubing, cannula/mask, oxygen outlet connector, and humidifier bottle. Replace cabin filter. Clean concentrator cabinet. Inspect plugs, cords and components. Check oxygen concentration.



#C4576

AIRSEP
A Chart Industries Company

- Lightest higher-capacity portable oxygen concentrator on the market at 4.4 lbs.
- Offers three pulse settings, so even more challenging ambulatory O₂ residents can exercise and have access to the oxygen they need

Specifications:	
Flow Rate:	0.5 - 3.0 lpm
Battery Life:	3 setting - 2 hours 2 setting - 2½ hours 1 setting - 3½ hours
O ₂ Purity %:	90% +5.5 / -3%
O ₂ Pressure:	5 psi
Power Consumption:	28W
Sound Level:	38 dBA at 1 setting; 41 dBA at 2 setting; 44 dBA at 3 setting
Dimensions:	6¼"W x 3¾"D x 8¾"H
Weight:	4.4 lbs.
Warranty:	3 years