

RESPIRONICS EVERFLO

5-LITER OXYGEN CONCENTRATOR



#96676

PHILIPS
RESPIRONICS

- Recessed flow meter and metal output valve improve durability
- Optional alarms for oxygen purity and flow rate
- Only two filters to change every 2 years – no weekly preventive maintenance

Specifications:

| | |
|--------------------------|--------------------|
| Flow Rate: | 0.5 - 5.0 lpm |
| O ₂ Purity %: | 90% - 96% |
| Outlet Pressure: | 5½ psi |
| Power Consumption: | 350W |
| Sound Level: | 45 dB |
| Dimensions: | 15"W x 9½"D x 23"H |
| Weight: | 31 lbs. |
| Warranty: | 3 years |

*2-year extended warranty available

Preventive Maintenance

Once a Year:



Verify the power-loss alarm is working. Let the concentrator run for 1 minute then unplug the power cord. An alarm should sound, indicating it is working properly.

Once Every 2 Years:



Check and, if necessary, change the inlet compressor filter. We also recommend changing between residents.



Check and, if necessary, change the micro disc bacteria filter. We also recommend changing between residents.



Verify the oxygen purity, flow rate and pressure of your concentrator by using an oxygen analyzer.

When changing residents:



Discard oxygen tubing, cannula/mask, oxygen outlet connector, and humidifier bottle. Replace cabin filter. Clean concentrator cabinet. Inspect plugs, cords and components. Check oxygen concentration.