



Powered by  
**DIRECT  
SUPPLY**

# DS smart® Chair Scale Tablet Mount Assembly Instructions

# Chair Scale with DS smart® Technology Tablet Mount Assembly Instructions

We have pre-assembled several components for you. Use these instructions to complete assembly of the chair scale tablet mount. To complete the assembly, you will need a  $\frac{5}{16}$ " wrench.



## Chair Scale Mount Assembly

- 1 Remove Indicator and Bracket:** Remove the indicator from the chair scale by pulling the lever underneath. Remove the bolts from the top of the column that hold the indicator bracket.



- 2 Attach the Tablet Holder to the Stand:** Align the 90-degree plate with the drill pattern from removed bolts. Replace the indicator bracket on top of the plate and install with the hex-head bolts provided in your mounting kit. Reattach indicator on top of the bracket.

**NOTE:** Do not reinstall the screws removed in Step 1, as it will cause instability of the tablet mount.



- 3 Position Tablet Mount:** Turn the handle on the tablet mount arm counterclockwise to loosen. Then, extend the arm to reposition the tablet mount, and turn the handle clockwise to tighten and fix in place.



- 4 Put Tablet into Tablet Holder:** If the tablet holder is closed, twist key in the lock to open the tablet holder. Insert the tablet into the tablet holder, push lever from left to right, lock and remove the key.



- 5 Attach the Stylus:** The stylus should be attached into the microphone slot at the top of the tablet.





## Wall Mount Assembly

Note: If you plan on keeping the tablet plugged in, affix the wall tablet mount near an outlet and, ideally, in a wall stud.

### 1 Gather Tools: For assembly, you'll need:

- Hammer
- Drill
- ¼" drill bit
- Phillips-head drill bit
- Four #6 screws
- Four anchors



### 2 Attach Wall Tablet Mount: Attach to wall using four screws. Then, resume assembly with Step 3 from previous page.





DS   
smart®

Powered by  
**DIRECT  
SUPPLY**

Distributed by Direct Supply Manufacturing, Inc.  
6635 N Industrial Road, Milwaukee WI 53223.

Direct Supply®, Panacea®, DS smart® and all associated logos  
are the trademarks of Direct Supply Inc., used under license.