



Powered by  
**DIRECT  
SUPPLY**

# DS smart<sup>®</sup> Hardware Assembly Instructions

for Edan iM3 Vital Signs Monitors

# DS smart<sup>®</sup>

## Hardware Assembly Instructions

We have pre-assembled several components for you. To complete hardware assembly, please use the instructions below.

### Tablet Case, Tablet Mount & Power Strip Assembly

You will need a small flathead screwdriver, a medium Phillips screwdriver and hex key (included) for assembly. Alternatively, you could use a power screwdriver/drill set. Please lock the wheels on the bottom of the vital signs monitor stand prior to continuing with assembly.




- 1 Attaching the Tablet Holder to the Stand:** Tablet mounting arm should be attached to the stand pole approximately 8" below the vital signs monitor using the hex key and screws. Tablet holder should face the front of the vital signs monitor. From bottom of tablet, tilt mount assembly all the way up, ending in a 45-degree angle. Then, lock in place by moving lever at top of tablet to the right.



- 2 Attaching the Power Strip to the Stand:** The power strip should face the back and the cord assembly the front. Top of power strip should be approximately 6" from the bottom of the basket and the plugs should be upside down.



- 3 Putting Tablet into Tablet Holder:** If the tablet holder is closed, twist the key in the lock to open the tablet holder. Push lever from left to right. Lock and remove the key and store in a safe place.

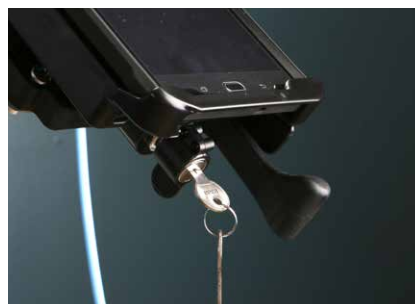



- 4 Connecting Equipment Cables:** Plug gray vital signs monitor power cord into the second outlet on the power strip. Plug in white tablet charger to power strip and connect white USB tablet cable to charger. Plug white USB tablet cable into top left of tablet.



- 5 Plug in Tablet Charger:** Plug the tablet charger into the power supply and feed the other end of the cable through the tablet case to prevent theft. To open the tablet case, use the lever behind the bottom of the screen.



- 6 Secure Tablet Charger:** Secure the tablet-charging cable by pushing down on the top and bottom of the case to close it. Be sure to plug the cable into the tablet with enough slack so as to not strain the cable at point of connection.



- 7 Attach the Stylus:** The stylus should be attached into the headphone jack at the top right of the tablet.



- 8 Attaching Cord Clips:** Secure the white tablet USB cable to stand pole using the three clips as shown. Note that the clips arrive with the vital signs monitor stand.





Distributed by Direct Supply Manufacturing, Inc.  
6635 N Industrial Road, Milwaukee WI 53223.

Direct Supply®, Panacea®, DS smart®, and all associated logos  
are the trademarks of Direct Supply, Inc., used under license.

