



Powered by  
**DIRECT  
SUPPLY**

**DS smart®  
Chair Scale  
Clamp Tablet  
Mount Assembly  
Instructions**

# Chair Scale with DS smart® Technology Clamp Tablet Mount Assembly Instructions

We have pre-assembled several components for you. Use these instructions to complete assembly of the chair scale tablet mount. To complete the assembly, you will need a  $\frac{5}{32}$ " Allen wrench, which is included in the hardware kit.

## Chair Scale Clamp Tablet Mount Assembly

- 1 Mount the Clamp Component:** Position the ball mount around the vertical tubing on the back of the chair scale. Position the height of the mounting kit for your expected use.

Place the included bolts through the ball mount, and attach the backing to the ball mount via the included hex nuts. Tighten the clamp using the  $\frac{5}{32}$ " allen wrench.



- 2 Position Tablet Mount:** Turn the handle on the tablet mount arm counterclockwise to loosen. Then, extend the arm to reposition the tablet mount, and turn the handle clockwise to tighten and fix in place.



- 3 Put Tablet into Tablet Holder:** If the tablet holder is closed, twist the key in the lock to open the tablet holder. Insert the tablet into the tablet holder, push lever from left to right, lock and remove the key.



- 4 Attach the Stylus:** The stylus should be attached into the microphone slot at the top of the tablet.



## Wall Mount Assembly

Note: If you plan on keeping the tablet plugged in, affix the wall tablet mount near an outlet and, ideally, in a wall stud.

- 1 Gather Tools:** For assembly, you'll need:
  - Hammer
  - Drill
  - $\frac{1}{4}$ " drill bit
  - Phillips-head drill bit
  - Four #6 screws
  - Four anchors



- 2 Attach Wall Tablet Mount:** Attach to wall using four screws. Then, resume assembly with Step 3 on previous page.





Powered by  
**DIRECT  
SUPPLY**

Distributed by Direct Supply Manufacturing, Inc.  
6635 N Industrial Road, Milwaukee WI 53223.

Direct Supply®, Panacea®, DS smart® and all associated logos  
are the trademarks of Direct Supply Inc., used under license.