



Powered by
**DIRECT
SUPPLY**

**DS smart[®]
Technology
Tablet Wall
Mount Assembly
Instructions**

Wall Mount Assembly

Note: If you plan on keeping the tablet plugged in, affix the wall tablet mount near an outlet and, ideally, in a wall stud. We have pre-assembled several components for you. Use these instructions to complete assembly of the tablet wall mount.

1 Gather Tools: For assembly, you'll need:

- Hammer
- Drill
- ¼" drill bit
- Phillips-head drill bit
- Four #6 screws
- Four anchors



2 Attach Wall Tablet Mount: Attach to wall using four screws.



3 Position Tablet Mount: Turn the handle on the tablet mount arm counterclockwise to loosen. Then, extend the arm to reposition the tablet mount, and turn the handle clockwise to tighten and fix in place.



4 Put Tablet into Tablet Holder: If the tablet holder is closed, twist key in the lock to open the tablet holder. Insert the tablet into the tablet holder, push lever from left to right, lock and remove the key.



5 Attach the Stylus: The stylus should be attached into the microphone slot at the top of the tablet.

